AirFit™

F40

Esnek Konforlu Minimal

Resmed AirFit™ F40

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*This mask contains magnets that may interfere with certain implants or medical devices.

Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

AirFit F40

Complete mask system

Product name	Product code
AirFit F40 System – SW/STD APAC	64645
AirFit F40 System – M/STD APAC	64646
AirFit F40 System – L/STD APAC	64647
AirFit F40 System – M/SML APAC	64648

Frame and vent ring

Product name	Product code
AirFit F40 Quiet Vent	64639
AirFit F40 Frame	64641

Full face cushion

Product name	Product code
AirFit F40 Cushion Small Wide	64630
AirFit F40 Cushion Medium	64631
AirFit F40 Cushion Large	64632

Headgear

Product name	Product code
AirFit F40 Headgear SML	64633
AirFit F40 Headgear STD	64634
AirFit F40 Headgear LGE	64635

Small in size. Big on features.

How to fit the AirFit F40

1 Unclip the lower straps away from the frame and place cushion under the nose to fit against the face.

With the Resmed logo facing up, pull headgear over head.

2 Bring the lower straps behind the ears and reattach to the frame with the magnetic clips*

Feed longer hair through the back of the frame, if needed.

3 Adjust the upper straps and then the lower straps for a snug fit.

Experience the Resmed AirFit F40 today

The minimalist ultra-compact full face mask is designed to deliver a reliable and adaptable seal. The soft, total silicone cushion moulds to the face and moves with patients as they sleep.

The Resmed AirFit F40. A minimalist mask, that adapts to your patient's natural sleep.

For more information, visit Resmed.com/AirFitF40 or contact your Resmed sales representative.

*This mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

¹ 97.7% Fit range based on data from two Resmed external clinical studies of experienced PAP patients. The first study included users with ≥ 6 months' therapy who used Resmed's AirFit F40 and AirFit F20 full face masks in randomized order for up to seven consecutive nights each at home. Study conducted in Australia, June 6 - 27, 2023, n=28 (AirFit F40) and 29 (AirFit F20). Facial measurements of 29 participants were taken. The second study included current users with ≥ 12 months' PAP therapy who met 90-day CMS compliance for PAP therapy in the three months prior to the study. Participants used Resmed's AirFit F40 and Fisher & Paykel's Evora Full in a randomized order with their own for up to seven consecutive nights each at home. Study conducted in the U.S., October 4 - November 20, 2023, n=57. Facial measurements of 57 participants were taken.

ALWAYS READ THE LABEL AND FOLLOW THE INSTRUCTIONS FOR USE. You should speak to your doctor about your symptoms and whether a CPAP device is suitable for you.

Resmed Pty Ltd
1 Elizabeth Macarthur Drive
Bella Vista NSW 2153 Australia

AirFit, AdaptiSeal and QuietAir are trademarks of the Resmed family of companies. For patent and other intellectual property information, see Resmed.com/ip. © 2025 Resmed Pty Ltd. 10114179/2 2025-10