Full Face Mask (F6S)

User Manual

CE0123

Full Face Mask (F6S)

Thank you for choosing BMC's Full Face Mask (F6S). The mask is comfortable, flexible and quiet. It is designed to minimize contact with your face, thus ensuring that you feel comfortable during therapy. This user manual provides you with the information you need for the correct use of your mask. The Full Face Mask (F6S) and accessories are not made with natural rubber latex or DEHP.

Intended use

The Full Face Mask (F6S) is intended to provide an interface for patients who have been prescribed CPAP or bi-level therapy. The Full Face Mask (F6S) is:

- to be used by adult patients (>66 lb/30 kg) for whom positive airway pressure therapy has been prescribed.
- intended for single-patient re-use in the home environment and multi-patient re-use in the hospital/institutional environment.

CAUTION: In the US, Federal law restricts this device to sale by or on the order of a physician.

Intended Operator

Lay users and professional personnel.

Contraindications

This mask should be used with caution for patients with following conditions:

- Facial deformity.
- Injured facial skin which will contact with the mask during use.

Before using the mask

⚠ WARNINGS


- The vent holes must be kept undamaged and free of blockages.
- The mask is not suitable for patients requiring life support ventilation.
- This mask should only be used with CPAP or Bi-level devices recommended by a physician or respiratory therapist.
- The mask should not be used unless the device is turned on. Once the mask is fitted, ensure the device is blowing air.
- Explanation:** When the CPAP or bi-level device is turned on and operating properly, the fresh air from the device flushes the exhaled air out through the attached mask exhalation port. However, when the device is not operating or at low pressure, the fresh air supplied through the mask is insufficient, and exhaled air may be re-inhaled. Rebreathing of exhaled air for more than several minutes may, in some circumstances, lead to suffocation. This applies to most models of CPAP or bi-level devices.

- To minimize the risk of vomiting during sleep, the patient should avoid eating or drinking three hours before using the mask. This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- Do not use the mask without the valve in place.
- Not applicable for silicone-allergy user.
- This mask should not be used on patients who are uncooperative, unconscious, unresponsive, or unable to remove the mask.
- Using the mask may cause pain in the teeth, gums or jaw or aggravate existing dental conditions. If these conditions occur, consult your health care professional.
- When the patient is unable to remove the mask himself, the mask must be used under qualified supervision.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bi-level device is not operating, so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- Oxygen supports combustion. Oxygen must not be used while smoking or in the presence of an open flame. Only use oxygen in a well-ventilated room.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration varies, depending on the pressure settings, patient breathing pattern, mask, point of application and leak rate.

- The technical specifications of the mask are provided for your clinician to check if it is compatible with the flow generator. If it is used under technical specifications or used with incompatible devices, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect the function of the flow generator.
- Stop using the mask and consult your physician or sleep therapist, if you have ANY adverse reaction to the use of the mask.
- Refer to your CPAP or bi-level device manual for details on settings and operational information.
- Remove all packaging before using the mask.
- Please wash your face before using the mask. Do not use moisturizer or lotion on your hands or face.
- Images shown here are indicative only. If there is inconsistency between the image and actual product, the latter shall govern.
- Any serious incidents relating to the product should be reported to BMC and to the competent authorities in your country.
- Frequency of cleaning, methods of cleaning or the use of cleaning agents, other than those specified in the accompanying documents, or exceeding the number of processing cycles can have an adverse effect on the mask and consequently the safety or the quality of the therapy.


- Stop using the mask and consult your physician or sleep therapist, if you have ANY adverse reaction to the use of the mask.
- Refer to your CPAP or bi-level device manual for details on settings and operational information.
- Remove all packaging before using the mask.
- Please wash your face before using the mask. Do not use moisturizer or lotion on your hands or face.
- Images shown here are indicative only. If there is inconsistency between the image and actual product, the latter shall govern.
- Any serious incidents relating to the product should be reported to BMC and to the competent authorities in your country.
- Frequency of cleaning, methods of cleaning or the use of cleaning agents, other than those specified in the accompanying documents, or exceeding the number of processing cycles can have an adverse effect on the mask and consequently the safety or the quality of the therapy.

Components of the mask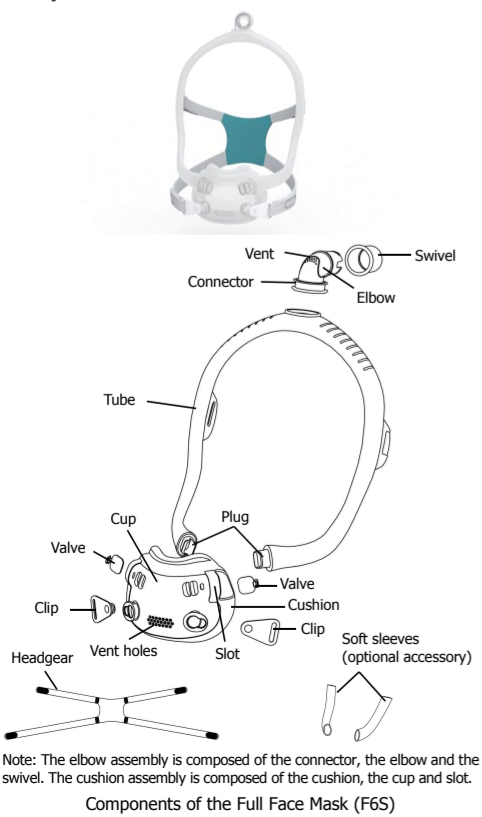

Note: The elbow assembly is composed of the connector, the elbow and the swivel. The cushion assembly is composed of the cushion, the cup and slot.

Choosing the right tube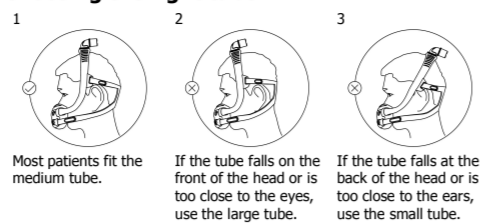


Most patients fit the medium tube.

If the tube falls on the front of the head or is too close to the eyes, use the large tube.

If the tube falls at the back of the head or is too close to the ears, use the small tube.


Fitting the mask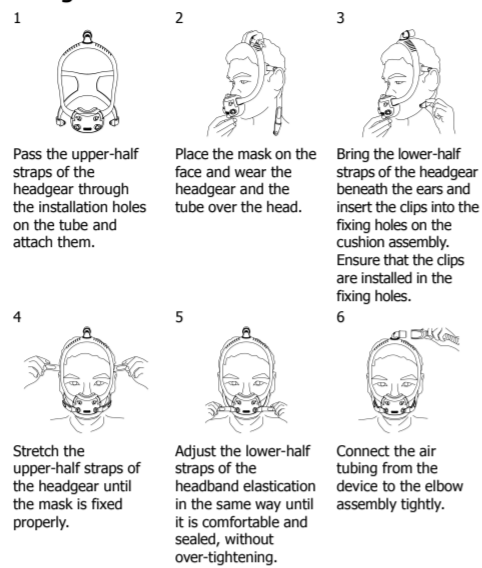


Disassembling the mask

Note:

- The connector cannot be disassembled from the elbow assembly.
- The valves cannot be disassembled. Or it may cause damage, and its installation would be difficult.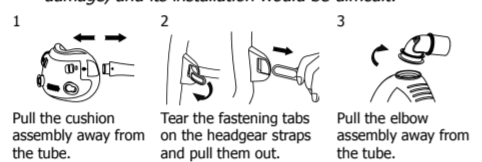

Reassembling the mask

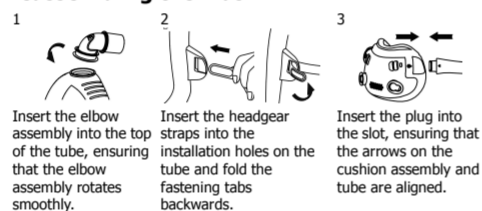Cleaning the mask at home

Notes:

- The cushion should be cleaned after each use.
- The headgear, tube and elbow assembly should be cleaned every week.
- The mask and headgear can only be cleaned by hand washing.
- Do not remove the connector from the elbow assembly when cleaning.
- Do not remove the valves from the cushion assembly when cleaning.

⚠ CAUTIONS

- Do not use solutions containing bleach, chlorine, alcohol, aromatics, moisturizers, antibacterial agents, or scented oils to clean any part of the mask. These solutions may cause damage and reduce the life of the product.
- Exposing any part of the mask to direct sunlight or heat may cause deterioration.
- Visual criteria for product inspection: If there is any visible deterioration of a mask component (cracking, crazing, tears, etc.), the component should be discarded and replaced.
- Replacement of parts should be carried out by professional service personnel.
- After cleaning, make sure the inside of tube is cleaned out.

Preparation before cleaning

- Prepare tools: soft bristle brush, drinking quality water (10 litres per mask), mild detergent (e.g. mild soap or Alconox).
- Disassemble the mask components according to the disassembly instructions.

Daily / After each use

- Soak the components in a solution of mild liquid detergent (e.g. 2% mild soapy water or 1% Alconox solution diluted with drinking water) (25°C to 35°C) for 10 minutes.
- While soaking, clean each component with a soft bristle brush for 1 minute. Pay special attention to all crevices and cavities.
- Agitate the components vigorously twice in drinking quality water. 5 litres of water per mask at a time.
- Allow the components to air dry and do not allow direct sunlight.
- When all components are dry, reassemble the mask according to the reassembly instructions.

Note:

- Cleaned once a day or after each use, the mask can be washed up to 365 times.

Weekly

Hand wash the headgear, tube and elbow assembly in a solution of mild liquid detergent (e.g. 2% mild soapy water or 1% Alconox solution diluted with drinking water) (25°C to 35°C) for 10 minutes. Rinse the headgear, tube and elbow assembly well with drinking quality water and allow them to air dry out of direct sunlight.

Note:

- Cleaned once a week, the headgear, tube and elbow assembly can be washed up to 52 times.

Reprocessing the mask between patients

This mask should be reprocessed when used between patients. Cleaning and disinfection instructions are available from the BMC website, www.bmc-medical.com. If you do not have internet access, please contact your BMC representative.

Note:

- The maximum number of disinfection cycles is 20.

Troubleshooting

Problem	Possible reason	Possible solution
Mask won't seal properly or is uncomfortable.	Mask may have been fitted incorrectly or is incorrectly adjusted.	Carefully follow instructions in "Fitting the mask" section. Make sure the headgear is not over-tightened.
	Mask size is wrong.	Consult your clinician.
	Mask is not tightened correctly.	Tighten both upper-half and lower-half headgear straps. Ensure cushion is properly aligned according to the instructions in "Reassembling the mask". Make sure there are no creases on the cushion.
	Cushion is misaligned or creased on face.	Clean cushion according to instructions in "Cleaning the mask at home".
Mask is too noisy.	Mask cushion may be dirty.	Clean cushion according to instructions in "Cleaning the mask at home".
	Mask is assembled incorrectly.	Reassemble mask.
Mask is too noisy.	Vent holes are blocked or partially blocked.	Clean the vent holes according to instructions in "Cleaning the mask at home" section.

Intended clinical benefit

- Improve patients' quality of sleep.
- Improve patients' compliance with therapy.

Storage

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.


Disposal

The mask does not contain any hazardous substances and may be disposed of with your normal household waste. For patients with transmissible diseases, the product shall be discarded as medical waste, otherwise it may cause the spread of disease.

Service life

The service life of the Full Face Mask (F6S) depends on the intensity of usage, maintenance, and environmental conditions to which the mask is used or stored. As this mask and its components are modular in nature, it is recommended that the user maintain and inspect it on a regular basis, and replace the mask or any components if deemed necessary or according to the "Visual criteria for product inspection" in the "Cautions" section of this manual. The mask should be replaced within 12 months from the date of first use.

Technical specifications

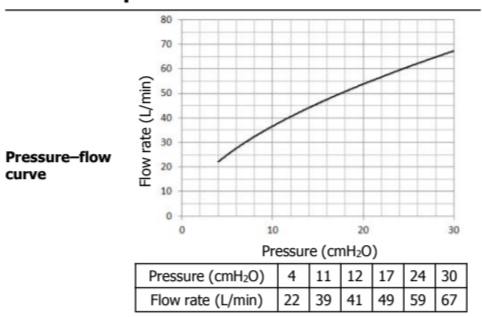Pressure-flow curve	Pressure (cmH ₂ O)
Flow rate (L/min)	4 11 12 17 24 30
Flow rate (L/min)	22 39 41 49 59 67

Dead space information	Dead space is the empty volume of the cushion and the tube up to the elbow assembly. The dead space of the F6S Large Cushion is 160 mL. The dead space of the F6S Large Tube is 140 mL.
Therapy pressure	4 to 30 cmH ₂ O
Resistance	Drop in pressure measured (nominal) at 50 L/min: 0.5 cmH ₂ O at 100 L/min: 1.5 cmH ₂ O
Inspiratory and expiratory resistance	The inspiratory resistance of the mask (in combination with the valve) is 1.4 cmH ₂ O at 50 L/min. The expiratory resistance of the mask (in combination with the valve) is 2.0 cmH ₂ O at 50 L/min.
Anti asphyxia valve open-to-atmo sphere pressure	< 4 cmH ₂ O
Anti asphyxia valve closed-to-atm osphere pressure	< 4 cmH ₂ O
Sound	DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 26 dBA, with an uncertainty of 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is 18 dBA, with an uncertainty of 3 dBA. Operating temperature: +5°C to +40°C (41°F to 104°F) Operating humidity: 0% to 93% relative humidity non-condensing Storage and transport temperature: -20°C to +55°C (-4°F to 131°F) Storage and transport humidity: 0% to 93% relative humidity non-condensing
Environmental conditions	
Date of manufacture	See product label

Symbols

System and packaging	Symbol	Description
Caution	⚠	Caution
Authorized Representative in the European Community	EC REP	Authorized Representative in the European Community
Batch code	LOT	Batch code
Catalogue number	REF	Catalogue number
Temperature limit	Temperature limit symbol	Temperature limit
Humidity limitation	Humidity limitation symbol	Humidity limitation
Manufacturer	Manufacturer symbol	Manufacturer
Use-by date	Hourglass symbol	Use-by date
Not made with natural rubber latex	No latex symbol	Not made with natural rubber latex
Importer	Importer symbol	Importer
Model number	#	Model number
Made in China	Made in China symbol	Made in China
Medical device	MD	Medical device
Unique device identifier	UDI	Unique device identifier

Limited warranty

It is warranted that the Full Face Mask (F6S), including its tube, cushion assembly, elbow assembly, clip and headgear shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer.

To exercise the rights under this warranty, contact the local authorized dealers.

MANUFACTURER:

BMC MEDICAL CO., LTD.
Room 110 Tower A Fengyu Building, No. 115 Fucheng Road, Haidian, 100036 Beijing, PEOPLE'S REPUBLIC OF CHINA
URL: en.bmc-medical.com
E-mail: intl@bmc-medical.com
Tel: +86-10-51663880

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Shanghai International Holding Corp. GmbH (Europe)